Montageanweisung [Mounting instruction]

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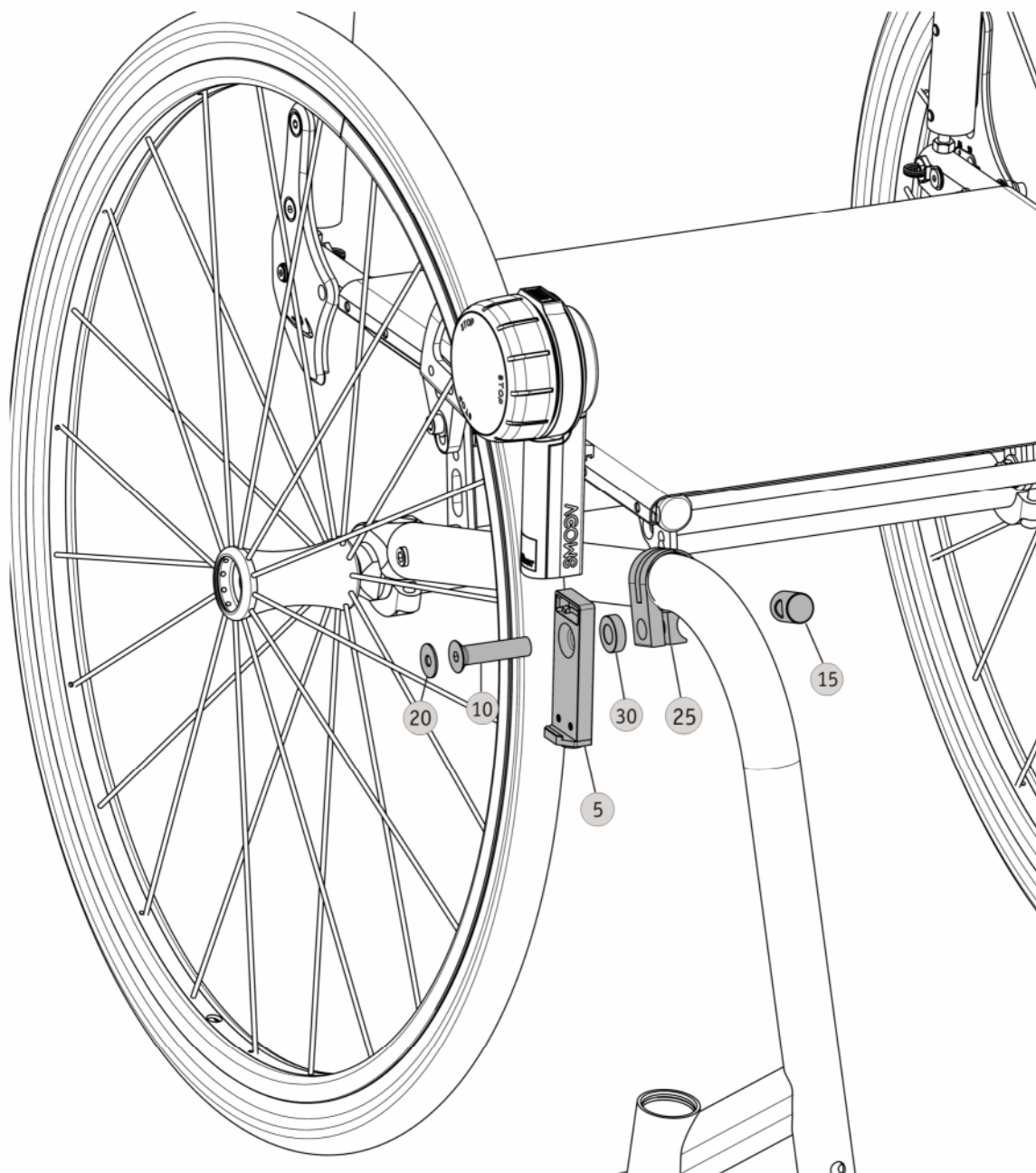
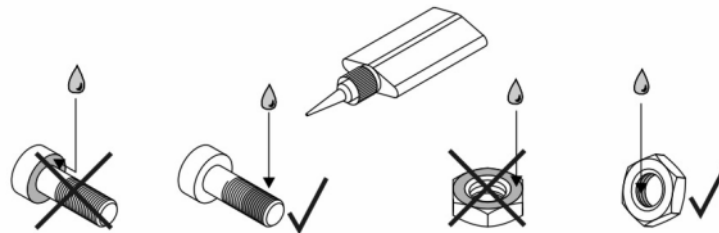
Locking of screws and screw nuts

If there is no other specification in our mounting instruction: Secure all screw fittings which have no locking washer with screw locking adhesive (see pictures below). Use WEICON adhesive AN 302-43 (blue) / LOCTITE 243 adhesive to glue adapter sockets and screws. Use WEICON adhesive AN 306-01 (green) / LOCTITE 601 adhesive to glue the mechanical fasteners of the e-fix control-unit.

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Allowed wheel camber of the e-fix and e-motion

Do not exceed the following maximum values, if you mount the bracket at a wheelchair which offers the

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Fastening torques for screw fittings

Unless otherwise noted, tighten the screw fittings as shown in the following table:

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Important Liability Note

A combination of the mounting systems introduced by Alber GmbH (Alber) together with add-on drives not manufactured by Alber is not authorized by Alber. Such combination do not comply with the intended use to be defined in accordance with the medical device directive (MDD) and therefore constitutes non-conformity according to the MDD.

By way of precaution, Alber GmbH rejects any liability regarding warranty services or possibly resulting property damages and personal injuries due to a combination of Alber mounting systems with third party brands.

Pos. [Pos.]	Menge [Quantity]	Einheit [Unit]	Art.-Nr. [Art.-No.]	Lagerplatz/Alber [Stock/Alber]	Bezeichnung [Description]
05	1	St	1592886	736/04/010	Halterung Bedieneinheit O10
10	1	St	1592842	736/04/009	Schraube Halterung Bedieneinheit
15	1	St	1593027	736/04/007	Nutenstein Halterung
20	1	St	1593028	736/04/001	Dämpfungselement
25	1	St	1593033	736/04/002	Klemme Profil 1
30	1	St	1592887	736/04/008	Distanzstück Halterung O10

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